


Reprocessing of Rigid Endoscopes

# Inspect, Assemble, Sterilize

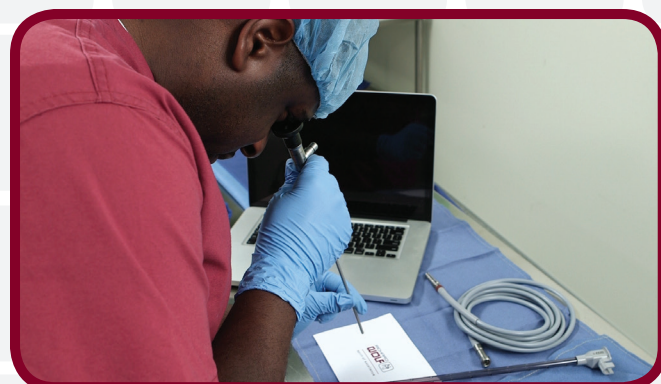

- ▣ Inspect optics
  - ▣ Clean residue from lens with alcohol swab
  - ▣ May dry channels with 5 cc 70% alcohol, filtered compressed air
- ▣ Inspect light bundle in scope—point scope up to ceiling light
    - >> Look directly into light post
    - >> Post should be milky white, little or no black spots
  - ▣ Can use portable light source to check
- ▣ Inspect light bundle in light cable
    - >> Hold both ends directly up to ceiling light
    - >> View end pointed up to ceiling light
    - >> Should be milky white, little or no black spots
  - ▣ Inspect light cable covering; plug ends
- ▣ Assemble scope accessories and attachments, place scope protector on scope
  - ▣ Sterilize according to IFU. Use primary sterilization modality.
  - ▣ Alternate reprocessing modalities at [www.richardwolfusa.com](http://www.richardwolfusa.com)